

Eye Wash Station Portable

EyeveX[®]
Our vision. Your safety

EEBS 1000



ISO9001 - EN46001



Spare Bottle 500 ml
EEB 500

Instructions

- To open pull lightly the transparent lid and it fall downward.
- Take out one of the bottles, twist the Eye Cup cap, the security seal will break enabling you to use the solution.
- The anatomic Eye Cup allows to direct the solution jet correctly, obtaining an efficient eye wash without any waste.
- After washing your eye, the mirror can be useful to find out the eventual damages occurred.

Eye Wash



- Contains the necessary items for an efficient First Emergency eye wash
- Thermo-formed, has a transparent lid
- Can be easily wall-mounted with screw and plugs, supplied in the kit

Contents:

- 2 Sterile Saline Solution of 500 ml
- 2 Integrated Eye cups
- 1 Mirror

Dimensions: 410mm X 320mm X 75mm

Solution Made in Germany
Packed in Italy